



JOINT JACKETS™

MANUFACTURED BY SOLARIS

Play Hard. Heal Fast.



Joint Jackets™

Thank you for choosing Joint Jackets.

Joint Jackets were developed from the same technology used to treat lymphedema. Solaris has spent the last four years developing unique products to help athletes recover faster from injuries.

Joint Jackets:

- Reduce inflammation, swelling, and pain
- Remove lactic acid formation and hematomas
- Allow for customized compression
- Provide support and joint protection
- Increase circulation and massage soft tissues
- Decrease scar adhesions

For more information on Joint Jackets
log on to www.solarismed.com



Table of Contents

Joint Jackets Components & Warranty	4
Washing Instructions	5
Joint Jackets Pro Units	6
Joint Jackets Bandage Liners	12

Joint Jackets Components

Pro Therapeutic Base Unit & Joint Jacket Bandage Liner

- Meryl/lycra fibers
- Long Lasting Velcro
- Latex-free
- Medical grade foam
- Manufactured in the USA

Pro Exterior Support Unit

- Nylon
- Lycra
- Elastane
- Polyurethane
- Long Lasting Velcro
- Latex-free
- Manufactured in the USA

Your Warranty

Joint Jackets Pro units are warranted to be free from material and manufacturer defect for 6 months from the date of purchase.

Washing Instructions

Therapeutic Base Unit & Joint Jackets Bandage Liner:

Machine wash the Therapeutic Base Unit in warm water on the permanent press cycle using a mild detergent. Dry on permanent press or lower heat setting. Heat exceeding 130 F will permanently damage the fabric and void your warranty.

Use of any additives, such as bleach or fabric softener, will void your warranty.

Exterior Support Unit:

Exterior Support Units should be hand washed as frequently as needed to eliminate dirt and odor.

Dip in warm water with a mild detergent, rinse thoroughly, and gently squeeze out excess water. Lay flat to dry. Machine washing and drying may shorten the useful life of your garment.

Pro Elbow Unit

Easy On:

Step 1 Gently slide the unit over the elbow. Take special care to ensure the jacket remains attached to the garment.

Step 2 Fasten the bottom strap.

Step 3 Fasten the top strap.

Step 4 Adjust both straps to ensure a snug and comfortable fit.



Pro Elbow Unit

Easy Off:

Step 1 Unfasten both straps. Do not detach the jacket from the garment.

Step 2 Gently slide the unit off of the arm, again making certain that the jacket remains attached to the garment.

Step 3 Store the unit in a cool, dry location.



Pro Knee Unit

Easy On:

Step 1 Gently slide the unit over the knee. Take special care to ensure the jacket remains attached to the garment.

Step 2 Fasten the bottom two straps.

Step 3 Fasten the top two straps.

Step 4 Adjust the straps to ensure a snug and comfortable fit.



Pro Knee Unit

Easy Off:

Step 1 Unfasten all straps. Do not detach the jacket from the garment.

Step 2 Gently slide the unit off of the knee, again making certain that the jacket remains attached to the garment.

Step 3 Store the unit in a cool, dry location.



Pro Ankle Unit

Easy On:

Step 1 Gently slide the unit over the foot. Take special care to ensure the jacket remains attached to the garment.

Step 2 Fasten the toe strap.

Step 3 Fasten the remaining straps, finishing at the top ankle strap.

Step 4 Adjust the straps to ensure a snug and comfortable fit.



Pro Ankle Unit

Easy Off:

Step 1 Unfasten the straps from the ankle to the toes. Do not detach the jacket from the garment.

Step 2 Slide the unit off of the foot, again making certain the the jacket remains attached to the garment.

Step 3 Store the unit in a cool, dry place.



Bandage Liner Elbow Unit

Easy On:

Step 1 Gently slide the garment over the elbow so the middle of the garment is over the elbow.

Step 2 Begin wrapping with a short-stretch bandage using a standard 50% overlap technique (shown right).

Step 3 Continue wrapping until the elbow has been sufficiently covered.



Bandage Liner Knee Unit

Easy On:

Step 1 Gently slide the garment over the knee so the the middle of the garment is over the patella.

Step 2 Beginning at the bottom of the garment, wrap the knee with a short-stretch bandage using a standard 50% overlap technique (shown right).

Step 3 Continue wrapping until the knee has been sufficiently covered.



Bandage Liner Ankle Unit

Easy On:

Step 1 Gently slide the unit onto the foot so that the toes and heel are exposed. Adjust for comfort.

Step 2 Beginning at the toes, begin to wrap with short stretch bandages. Be sure to use a 50% overlap technique (shown right).

Step 3 Continue wrapping until the ankle has been sufficiently covered.



SOLARIS

F R E E D O M T O L I V E

Solaris, Inc.

6737 West Washington Street, Suite 3260

West Allis, WI 53214

Client Hotline: 888-918-9185

T: 888-918-9180

F: 414-918-9189

E: info@solarismed.com

www.solarismed.com